

## **Aquacut Daily Start-up Routine**

1. Make sure the airline is connected to the back of the Aquacut.
2. Place a new tip onto your hand piece.
3. Put fresh distilled water into the Aquacut's water reservoir.
4. At this time make sure there is no powder in either dosing chamber.
5. Turn the machine on to position 1.
6. Ensure the needle on your ABC meter, on the front of your machine, rests between B and C. If not, adjust the SPEED control to make it so.
7. You should not hear any air leaks at this time. If you do hear air leaking from the machine, turn your machine off and check the dosing chamber lids for tightness. Turn your machine back on. Listen again for any air leaks. Eliminate any air leaks before continuing.
8. Push the foot pedal down to the furthest position, which is position 3. Keep it depressed until you see a good flow of water and air from the hand piece. This can take approximately 30 to 40 seconds.
9. Turn the machine off.
10. Put the powder of your choice into the dosing chamber/chambers.
11. Turn your machine back on to position 1.
12. Your machine is now ready for use.