AquaCare Twin

Dental Air Abrasion & Polishing Unit

AT100000

Quick Start Guide

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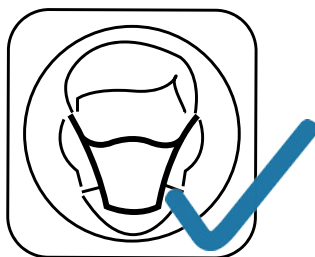
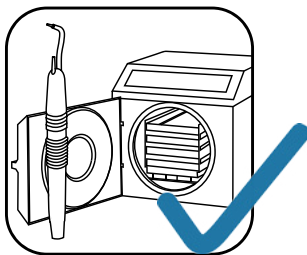
PROTECT SENSITIVE EQUIPMENT FROM DUST

Protégez les équipements sensibles de la poussière
\ Schützen empfindliche Geräte vor Staub
\ Proteggere le apparecchiature sensibili da polvere
\ Proteja el equipo sensible al polvo
\ Защитите оборудование от пыли

STERILISE HANDPIECE BETWEEN TREATMENTS


Stérilisez la pièce à main entre les traitements
\ Sterilisieren Sie das Handstück zwischen den Behandlungen
\ Sterilizzare il manipolo tra i trattamenti
\ Esterilice las piezas manuales entre tratamientos
\ Стерилизуйте наконечник перед каждым приемом

DON'T STEP ON TUBES

Ne marchez pas sur les conduites
\ Treten Sie nicht auf Leitungen
\ Non calpestare i tubi
\ No pise los tubos
\ Не наступайте на трубки

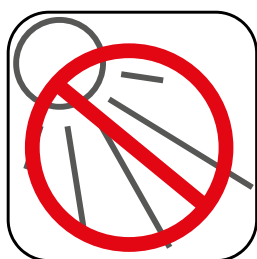DON'T PLACE IN SUNLIGHT


Ne l'exposez pas à la lumière directe du soleil
\ Vermeiden Sie direkte Sonneneinstrahlung
\ Non esporre direttamente alla luce del sole
\ No lo exponga al sol
\ Избегайте попадания прямых солнечных лучей

REPLACE TIP BETWEEN TREATMENTS

Remplacez l'embout entre les traitements
\ Tauschen Sie die Spitze zwischen den Behandlungen aus
\ Sostituire i puntali tra i trattamenti
\ Reemplace las puntas entre tratamientos
\ Меняйте пластиковые насадки перед каждым приемом

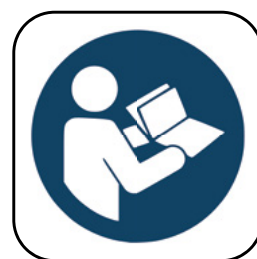USE ONLY VELOPEX PARTS

Utilisez exclusivement des pièces Velopex
\ Verwenden Sie nur Velopex-Ersatzteile
\ Usare i componenti Velopex
\ Utilice únicamente piezas Velopex
\ Используйте детали Velopex

Uses

USES

(GB)

Preparation of pit and fissure for sealants. Removal of composites for restoration. Cavity preparation. Cleaning, polishing and stain removal. Etching.

ONLY QUALIFIED DENTAL PERSONNEL SHOULD USE THE AQUACARE. GIVE SPECIAL CONSIDERATION TO PATIENTS WITH:

- Respiratory difficulties
- Asthma
- Blocked nasal passages
- A cold
- Restricted sodium diet
- Any medical problem preventing breathing through the nose

Patients suffering from the above must consult their physician before treatment.

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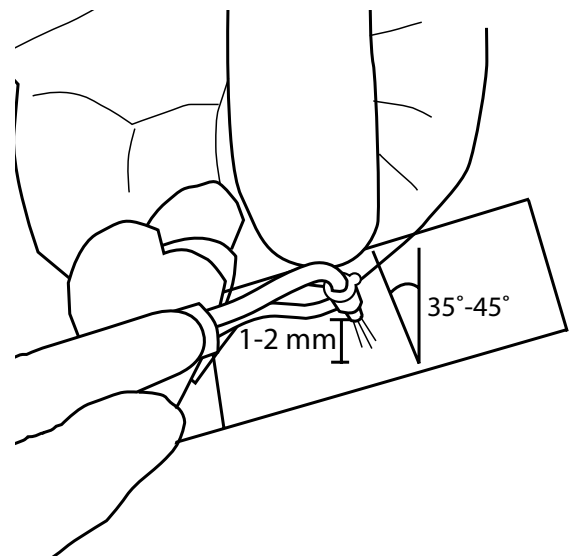
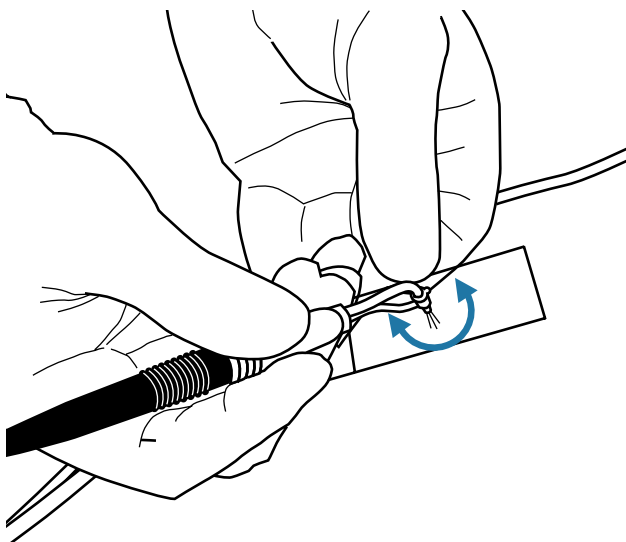
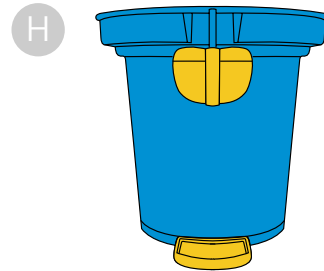
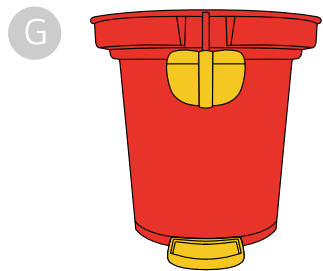
Suggested Treatment Methods

The following treatment suggestions have been prepared with the help of experienced practising dentists. The methods of treatment are advisory, and each dentist will quickly develop their own preferences in terms of powersetting, powder type, media volume, fluid volume, and the angle of the cutting nozzle.

Treatment Tips

The end of the cutting nozzle should be held approximately 1.5mm from the working surface. Avoid holding the nozzle at a right angle to the surface, as this will cause the powder to rebound and blur the view of the area. Angle the cutting tip towards the aspirator and instruct the Dental Assistant to follow the nozzle with the aspirator. The AquaCare operates with low noise, and has no vibration or heat at the cutting tip. Its ability to cut wet or dry gives it a high flexibility of use. Local anaesthetic can be offered to nervous patients, however it is generally not required as air abrasion is painless in most cases. The various treatments use a range of settings. The AquaCare has three dynamic controls which alter the cutting speed/cleaning rate. These are the power setting, the media volume, and the fluid volume. By increasing one setting as you decrease another, you can maintain the same cutting speed/cleaning rate but alter the treatment provided. Use less media and more power to improve viewing of the working area. Use more media and less power for sensitive areas.

Cleaning Accessory

The AquaCare Dental Air Abrasion Cabinet offers a simple aid in support of your AquaCare abrasion unit to enable abrasion procedures to be done outside of the mouth. Preparation of dental attachments such as crowns, bridges or brackets can be cleaned and abraded before they are cemented in the mouth.

Preparations Before Treatment

WARNING: All people in the operating area must wear eye protection to BS EN 166 IF 415

WARNING: All people in the operating area except for the patient must wear respiratory masks to EN 149 FFP2S

1. It is recommended that a light coating of petroleum jelly is smeared around the patient's lips to prevent cracking and to retain some of the outflowing abrasive powder.
2. Rubber dam should be used to protect adjacent teeth and soft tissue.
3. Non-metal Matrix strips should be used to protect adjacent teeth during interproximal preparations.
4. A bib should be used to protect the patient's clothes from any out-flow of the material from the mouth during the course of a treatment.
5. Use a high-speed intra-oral aspirator, preferably with a flared tip.
6. An extra-oral evacuator can also be used with the extraction cone sited below the patient's chin.
7. **IMPORTANT:** To avoid unnecessary damage, when switching from Aluminium Oxide to Sodium Bicarbonate you must purge the powder.
8. It is recommended that AquaCare Cleaning/Cutting Fluid AquaSol is used with the AquaCare Fluid System. The 500ml bottle supplied with the machine should be discarded when empty and replaced. The fluid can be used with all recommended treatments using this instrument.

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Dental Air Abrasion & Polishing Unit

Recommended Accessories

WARNING: All replacement parts and consumables must be sourced from Velopex to ensure correct and safe functioning of the AquaCare.

Description

Equipment

Breathing air filter unit
6mm Input Air Hose
AquaCare Stand (Patient Treatment Centre)
Velopex Dental Air Abrasion Cabinet

Part No

I/FIT 8200F
I/ROL 8014F
I/ASS 0045F
I/MAC 6200F

Consumables

Sylc Bioactive Powder Cartridges, Pack of 4
Sodium Bicarbonate Cartridges, Pack of 4
29 micron Aluminium Oxide Cartridges, Pack of 4
53 micron Aluminium Oxide Cartridges, Pack of 4
Glass Slides (for cutting practice), Pack of 5
Feedlines and Tips, Pack of 50
Handpiece Handles, Pack of 5
Handpiece Tube Removal Tool
Handpiece - 0.4mm Green
Handpiece - 0.6mm Silver
Handpiece - 0.8mm Gold
500 ml Cutting Fluid Bottle (AquaSol)
Nozzle Cleaning Wire
Nozzle Wash Syringe, Pack of 2

I/PDR 0034F
I/PDR 8014F
I/PDR 8025F
I/PDR 8024F
I/MIC 0010F
I/ASS 8007F
I/ASS 8024F
I/FIT 0025F
I/ASS 8189F
I/ASS 8191F
I/ASS 8192F
I/PAC 8200F
I/FIT 8073F
I/ASS 8025F

Spare Parts

Storage Tank and Lid
Dosing Chamber Lid
Powder Storage Pot
Twin Tube (with check valve)
Check Valve, Pack of 2
Foot Pedal Assembly
Dosing Chamber Starfish Seal

I/ASS 8193F
I/ASS 8003F
I/ASS 8197F
I/ASS 8020F
I/ASS 8004F
I/FIT 8153F
I/FIT 0012F